

Needle holder with Tungsten Carbide Inserts



Check jaws are 'seated correctly' and there are no obvious gaps under the insert



Ensure the jaws do not overheat during repair - this causes damage to the needle holder



Check box joint for cracks or damage
Ensure box joint is not stained or rusted inside




Lock ratchet/rack and drop onto a soft surface from 30cm to prevent jumping off of the rack




Gold bows indicate Tungsten carbide jaws




Check the Tungsten carbide inserts are not worn

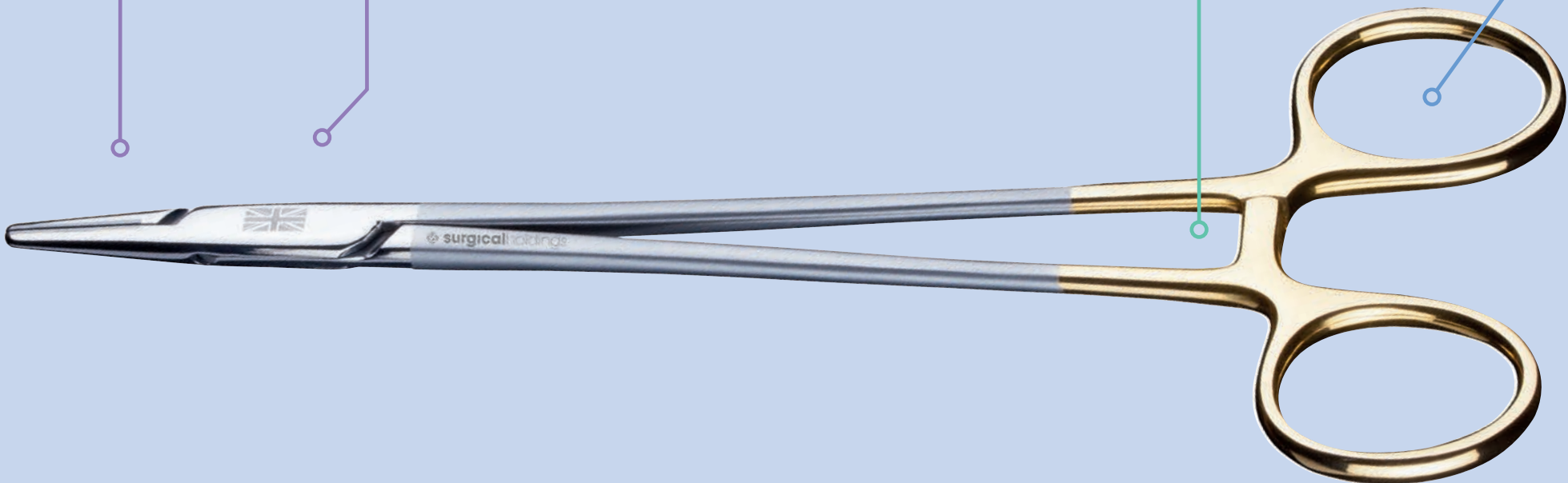


Each needle holder jaw is designed for a specific needle:

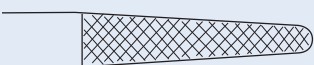


Coarse 0.65mm

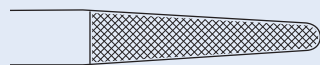


Standard 0.5mm

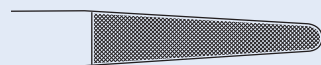

Fine 0.4mm

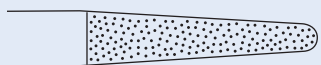

Micro 0.2-0.3mm

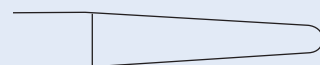

Smooth