



Check blades of Mayo, Metzenbaum and McIndoe scissors are smooth and not damaged

Look for damage or rust along the blades



Check for cracks or damage around the screw areas



Ensure scissors have a smooth action




These scissors must be maintained and sharpened correctly or they can be easily damaged beyond repair



One blade has a super fine serration and the other a razor edge

Super Cut blades

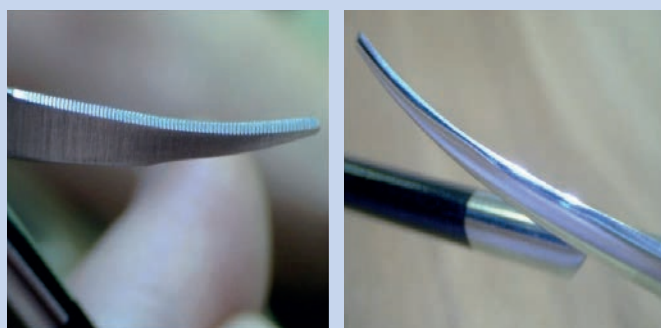

Check the scissors cut correctly using a test material recommended by your manufacturer



Look out for 2 black bows which designates 'super cut edge'

Black bows are only generally a plating so may scratch



Some scissors may appear to be looser but set is correct depending on scissor type; so check with manufacturer if unsure